






## Face Shield Assembly Instructions

1	Remove tubes from bag, then peel completely apart.	<div data-bbox="905 314 1564 363">SCAN FOR ASSEMBLY VIDEO</div> <div data-bbox="968 387 1509 436"><a href="https://youtu.be/6lOzl9hVRgl">https://youtu.be/6lOzl9hVRgl</a></div> <div data-bbox="1042 529 1352 843"></div>
2	Press end of tube into pointed end of push pin.	
3	Ensure tube is fully seated against plastic (repeat for both sides).	
4	Face shield complete.	